NOTICE FABRICANT / INFORMATION SUPPLIED BY MANUFACTURER

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/ Filter solid and liquid particles suspended in air with filtration efficiency over 94%

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/ Check the product validity by ensuring the use is made within the shelf life date. If it expires, discard it immediately. Wrong use of this product can result in illness or death.

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EN

Make sure to wash your hands before putting on the mask. Once in place, do not handle the mask as there is a risk of deterioration of it and contamination of the hands. Practice simple washing or hydro alcoholic rubbing of the hands before and after each change of mask.

1. Hold the mask by the earloops with the nosepiece at the top.
2. Press the mask against your face with the nosepiece on the bridge of your nose.
3. Place the earloops behind your ears.
4. Pinch the nosepiece with both hands to adjust it to the level of the nose. Lower the bottom of the mask under the chin. Respect the laying direction (nosepiece at the top). The mask should be worn covering the nose, chin and mouth. It should be applied tightly to the face.
5. To check the correct fit of the mask, mold the nosepiece to the shape of your nose using both hands. Test the fit. Place both hands on the respirator and breathe out vigorously. If air is circulating around your nose, tighten the nosepiece. If air leaks around the edges, reposition the ear loops for a better fit. Remove and/or replace the mask immediately if/when a correct face fit cannot be maintained.

After use, remove the mask by grabbing the elastic from the back without touching the front part of the mask. Single-use masks should be discarded immediately after each use in a bin with a plastic bag (preferably with a lid and not manually operated) as it is not possible to decontaminate them. Wash your hands or apply friction with a hydroalcoholic solution after removal.

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EN

Keep this information for future reference.
This filtering half mask to protect against particles filter solid and liquid particles suspended in air with filtration efficiency over 94%. As requested by World Health Organization recommendations, the nominal protection factor given by this filtering half mask FFP2 NR D is defined in EN 149:2001+A1:2009.
Warning - this product is not flame resistant and must not be used in areas with open flames. This half-mask does not provide oxygen. Do not use in an explosive atmosphere. Do not use it in low oxygen atmospheres.
Do not remove the mask from its packaging until you are ready to use it. Before use, always check the condition of the mask and its attachments. If the checks prior to use are unsatisfactory, do not use the product, dispose of it and use another one.
The mask is intended for use by only one person. An increase in breathing resistance may indicate that the mask is clogged. Remove and/or replace the mask immediately if you experience difficulty in breathing, nausea, discomfort, dizziness or other distress or when the mask is damaged / deformed, or a correct face fit cannot be maintained.
Warning: facial hair (beard, mustache, sideburns, etc.) and glasses or goggles can hinder the correct proper fitting of the mask and reduce its effectiveness.
This disposable mask does not require maintenance, disinfecting nor cleaning. It is designed for one shift only (i.e. a single continuous period of work). The usable time of the mask may differ according to the national recommendations of each country. Throw it away after just one shift, following the instructions. Failure to follow the instructions and warnings regarding the use of this filtering half mask to protect against particles during the entire exposure period may reduce the effectiveness of the latter. This product does not eliminate the risk of contracting diseases or infections. Improper use of this mask can result in illness or death.

CONDITIONS DE STOCKAGE / STORAGE CONDITIONS

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EN

To keep their properties, unused masks should be stored in the original packaging, in a clean, well-ventilated, dry and uncontaminated place, with a temperature between -30 and +70 °C and a relative humidity below 80% until used. The shelf life of unused masks is 3 (three) years from the date of manufacture if storage conditions are met.

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